



RosieConnectivity SOLUTIONS™

Cleaning & Disinfecting Your Rosebud^{VC} Monitor

Clean your equipment on a regular basis. Consult your facility's regulations before cleaning the equipment.

The recommended cleaning agents include:

Sodium hypochlorite bleach (10%) and Isopropanol (70%)

- ◆ Shut down the monitor and disconnect it from the AC power.
- ◆ Clean the display screen using a soft, clean cloth dampened with a cleaning agent.
- ◆ Clean the exterior surface of the equipment using a soft cloth dampened with a cleaning agent.
- ◆ Wipe off all the cleaning solution with a dry cloth after cleaning if necessary.
- ◆ Dry your equipment in a ventilated, cool place.



Disinfecting

- ◆ If disinfecting is required because of your facility's policy, cleaning the equipment before disinfecting is recommended.
- ◆ The recommended disinfectants include:
 - ◆ Ethanol (70%)
 - ◆ Isopropanol (70%)
 - ◆ Perform® classic concentrate OXY
- ◆ Disinfecting Temperature Probe
- ◆ Disconnect the temperature probe from the Temp connector.
- ◆ Disinfect the probe with a soft cloth dampened with the recommended disinfectant.
- ◆ Wipe off all the remaining disinfectants from the probe with a soft cloth dampened with water.
- ◆ Dry the probe in a cool place.

Tips

- ◆ Always dilute cleaners and disinfectants according to the manufacturer's instructions or use the lowest possible concentration.
- ◆ Do not immerse any part of the equipment into liquid
- ◆ Do not pour liquid onto the equipment or accessories
- ◆ Do not allow liquid to enter the case.
- ◆ Never use abrasive materials, or corrosive cleaners (such as acetone or acetone-based cleaners.)

⚠ WARNING

- ◆ Be sure to turn off the monitor and disconnect all power cables from the outlets before cleaning the equipment.
- ◆ Use only Rosie Connectivity Solutions approved cleaners and disinfectants and methods to clean your equipment.
- ◆ Warranty does not cover damage caused by unapproved substances or methods.
- ◆ We make no claims regarding the efficacy of the listed chemicals or methods as a means for controlling the infection.

⚠ CAUTION

- ◆ If you spill liquid on the equipment or accessories, contact Rosie Connectivity Solutions or your service personnel.