



# Rosebud<sup>®</sup>VC

## CALIBRATION PROTOCOL



- 1 - Turn the Rosebud<sup>®</sup>VC off. The power button should be black not orange.
- 2 - Turn the Rosebud<sup>®</sup>VC back on. In about 10 to 15 seconds you will hear a “beep”.
- 3 - After the “beep”, press and hold the “C” button down until you see **NIBP** on the screen. Let go of the “C” button.
- 4 - Tap the “C” button, and you will see **TEMP** at the bottom of the screen. If you need to change the unit of measure tap the **BLOOD PRESSURE** button once.
- 5 - Tap the “C” button again, and you will see the TIME SET-UP. If you need to change the time, tap the BLOOD PRESSURE button to change to number value. To move to the next digit, press the **ADULT, CHILD, INFANT** button to move to the next digit.
- 6 - Tap the “C” button 5 times or until you see “OFF” on the right side of the screen. Tap the **BLOOD PRESSURE** button once to change the reading to “ON”.
- 7 - Push the “C” button until you are back to **TEMP** at the bottom of the screen. Change the setting to **FAHRENHEIT** measure by tapping the **BLOOD PRESSURE** button once.
- 8 - Turn the Rosebud<sup>®</sup>VC off. The power button should be black not orange.
- 9 - Turn the Rosebud<sup>®</sup>VC back on, and wait 15 seconds after the “beep” to take your first reading. After the reset, you may need to take a second reading before the temp registers correctly.

**NOTE:** Always check the *Temperature Probe Well* for obstructions (probe covers, etc.) as this can affect temperature readings.