



Rosie® Standard Cart

Assembly Instructions

Before beginning assembly, inspect the product for any damage. If there is damage to the product, stop and take photos of the damage and the shipping container. Contact Rosie immediately at 1-800-841-1109.

Tools required: Philips Screwdriver, Allen Key (size 4)

- ✦ Place base on the floor.
- ✦ Assemble and Mount Basket:
 - ✦ Affix basket mounts to basket as shown in printed instructions included in basket packaging.
 - ✦ Take base pole and slide the basket up from the bottom to the middle of the pole.
 - ✦ Tighten screws to hold basket in place.
- ✦ Install pole
 - ✦ Place cap over the bottom of the pole and insert into the base.
 - ✦ Place washer against bottom of the base and screw base to pole until completely tight.
- ✦ Mount Tympanic Thermometer
 - ✦ Use Allen wrench to loosen the screws on the Rosie SmartTemp Tympanic Thermometer mount.
 - ✦ Then place on each side of the pole (about 8 inches from the top) and tighten into place.
- ✦ Install Vital Signs Monitor
 - ✦ Install battery by opening the battery door at the bottom of Vital Signs Monitor. Insert battery and close.
 - ✦ Remove and retain screws and metal back plate that comes pre-installed on the mount at the top of the pole. Place monitor on top, making sure the hole lines up with the pegs on the metal base. Reattach metal back plate with original screws to hold monitor in place and tighten screws.
- ✦ Install Cart Handle
 - ✦ Unscrew end piece. Place on each side of the pole and screw into place.
- ✦ Plug power cord into the back of the monitor.
- ✦ Install Oral Thermometer
 - ✦ Insert the blue oral probe well in the top of the Vital Signs Monitor.
 - ✦ Attach Oral Temperature probe by sliding the blue holder into top right of the monitor until it clicks into place.
 - ✦ Put the oral probe into the oral probe well.
- ✦ Install BP Hoses and Cuffs
 - ✦ Attach NIBP Hose to the left side of the monitor.
 - ✦ Take your preferred cuff size and lock it into the hose. Pull back quick-release tab to remove. **DO NOT** pull apart without releasing the quick-release as this will cause damage.
- ✦ Install SPO2 Sensor
 - ✦ Attach SPO2 cable to the left side of the monitor, making sure to line the arrows up.
 - ✦ Take the finger cuff attachment and click the blue end into the cable. Close the clear cover over the connected pieces.

