


How to clean your Oximeter (SPO2 Sensor)

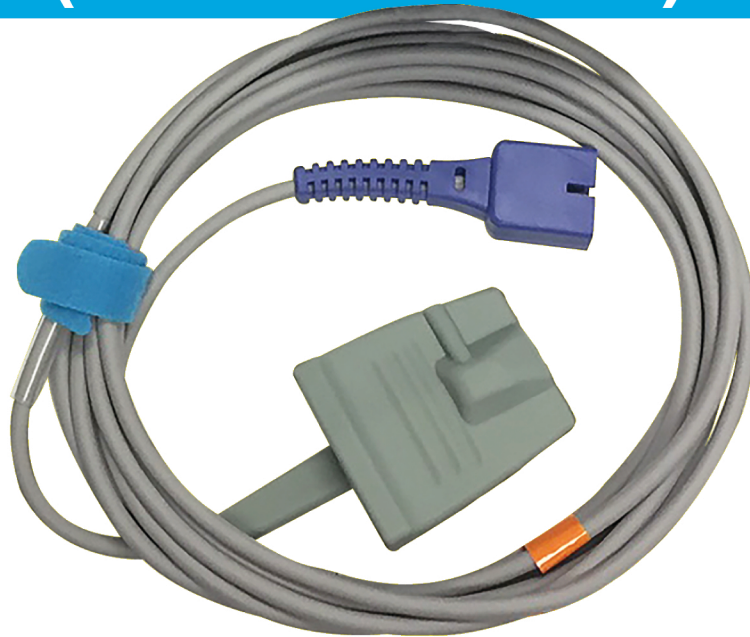

- ◆ The inside of your SPO2 oximeter can be cleaned using a soft, dry cloth. Avoid the use of water or liquid to clean the LEDs and sensors, as this can damage them.
- ◆ To clean the outside (or the shell) of your SPO2 oximeter you can use a damp, clean cloth or wipe to disinfect.
- ◆ Wipes containing sodium hypochlorite as the active ingredient and gauze soaked in bleach (1:10) have been shown to be the most effective in removing both vegetative bacteria and spores.
- ◆ Do not over saturate the cloth to avoid causing damage to the LEDs and sensors.
- ◆ Be extremely gentle when cleaning both the inside and outside of the SPO2 oximeter.